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For more information, please contact:

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User guide - Earplugs for sleeping

EN - User guide - Earplugs for sleeping

The hearing protection is custom-made according to the ear impression. Please read the instructions for use carefully before starting to use your hearing protection.

Usage

Insertion : Grasp the hearing protection so that the coloured dot is facing upwards. The red dot corresponds to the right side and the blue dot to the left side. Insert the hearing protection into the ear canal (inner part of the ear). Make sure that all parts of the hearing protection are correctly placed in the folds of the outer part of the ear.

Before inserting the hearing protection into your ear, we recommend that you turn it a quarter turn forward. Then turn the hearing protection a quarter turn backwards so that it is correctly positioned in the ear canal. **Removal :** Grasp the hearing protection to remove it. We recommend that you turn the hearing protection a quarter turn forward to remove it.

Noise tightness

Always place the hearing protection so that the ear canal is properly blocked. Position the hearing protection correctly so that the noise reduction is guaranteed.

Storage and Maintenance

The hearing protection is reusable. It should be cleaned after each use. Cleaning with soap and water. Chemical detergents and compressed air should be avoided as they are harmful to the product. Never heat the hearing protection to dry it. Let it dry at room temperature. The life of the hearing protection depends on the use and maintenance. We recommend storing the hearing protection in the case included in the delivery for storage and transport.

Skin irritations and dysfunctions

The hearing protection is made of hypoallergenic and biocompatible medical silicone material. It is coated with a hypoallergenic lacquer. Consult an ENT specialist in case you still have itching or allergies. This may be an over-reaction of the skin against the material used. Please contact your dealer or the manufacturer in the event of incorrect positioning of the protection or if the sealing effect of your hearing protection is insufficient. If the hearing protection is inserted incorrectly or if the instructions for use are not followed, the effectiveness of the hearing protection may be impaired. A possible change is free of charge for the first 3 months.

Warnings

The hearing protection is for personal use only and should not be worn by another person. The retailer and the manufacturer accept no liability for a possible reduction in noise tightness. They also decline all responsibility for the consequences of wearing hearing protection. The hearing protection cannot be used as part of a noise protection system. Hearing protection is not designed to protect the ear from water. Sleep hearing protection is considered to be comfort hearing protection and is not subject to personal protective equipment certification. Contact the manufacturer or your dealer for further information. It is the responsibility of the user to check the suitability for the intended use, taking into account any influences that may affect performance. Warranty and liability claims for our products are provided for in the general terms and conditions of sale, unless otherwise stipulated by law.